

# STERILE SPECIMEN COLLECTION SWAB


Honey Dipper Nasopharyngeal Swab  
with 70mm Breakpoint



As we navigate through these unprecedented times, Applied Medical is committed to rapidly responding to the needs of the healthcare community. We are all in this together, fighting against COVID-19.

# HONEY DIPPER NASOPHARYNGEAL SWAB

## Ct VALUES OF SARS-COV2 POSITIVE HUMAN SPECIMEN

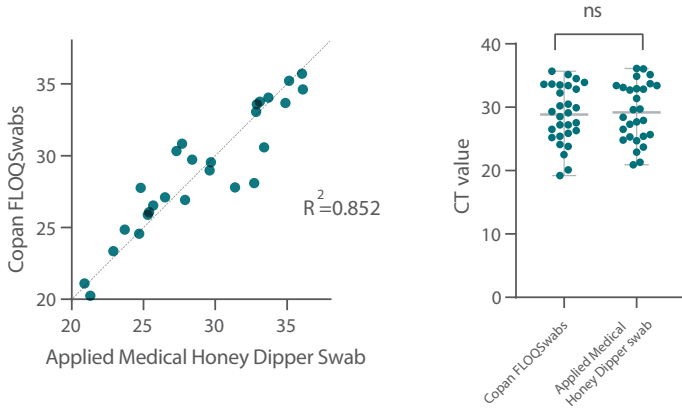

The Applied Medical Honey Dipper swab was validated in COVID-19 positive patients at a major academic center and yielded comparable results to the Copan FLOQSwabs™.\*

Linear regression analysis and paired t-test of the yielded data showed that the Applied Medical Honey Dipper swab is comparable to the Copan FLOQSwabs. The amount of virus that both swabs collected is correlative and statistically no different from each other.

Honey Dipper Tip

Breakpoint at 70mm from Tip

5.9"

	Applied Medical Honey Dipper Swab (NPS19)	Traditional Flocked Swab
Tip Material	Polystyrene	Polyester
Tip Diameter	3.55mm	3.56mm
Breakpoint	70mm	80mm
Shaft Material	Polystyrene	Polystyrene
Length	5.9"	6.0"

## FEATURES

- Sterile, individually packaged swabs
- Made in the USA with materials sourced domestically

## ORDER INFORMATION

Model #	Description	Qty/Box
NPS19	Sterile specimen collection swab	200*

\*Controlled by weight

\* Data on file

Copan FLOQSwabs is a trademark of Copan Diagnostics Inc.

© 2020 Applied Medical Resources Corporation. All rights reserved. Applied Medical, the Applied Medical logo design and marks designated with a ® are trademarks of Applied Medical Resources Corporation, registered in one or more of the following countries: Australia, Canada, Japan, South Korea, the United States, and/or the European Union. 303732-EN-USA-C

