

# Kii® Dissecting Balloon Access Systems

## SETUP GUIDE



**1** Insert introducer until introducer and seal are locked into place.



**2** Insert dissecting balloon into incision site with inflation port perpendicular to the abdomen.



**3** Remove introducer.




**4** Insert 10mm endoscope.



**5** Connect the inflation bulb and twist the Luer lock connector clockwise onto the stopcock until firmly attached.



**6** Inflate the dissecting balloon under visualization by compressing the inflation bulb.

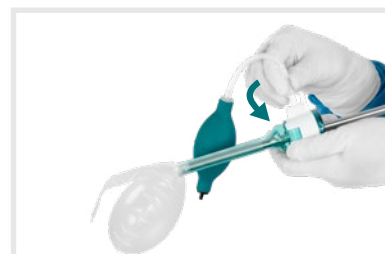

**7** After completing dissection, untwist the Luer lock connector counterclockwise and disconnect the inflation bulb.



**8** Remove the endoscope for complete balloon deflation.

Visit [appliedmedical.com/dissectingballoons](https://www.appliedmedical.com/dissectingballoons) for more information.

This information is not intended to replace the Instructions for Use (IFU). Please refer to the IFU for the indications, contraindications, warnings, precautions, instructions and other information.